



IALURIL® Ovules/IALUNA

Ialuril® Ovules contain sodium hyaluronate.

Composition

Each 2 g ovule contains:

Sodium hyaluronate, fat-soluble extract of Centella Asiatica, fat-soluble extract of Chamomilla vulgaris, d,l-alpha tocopheryl acetate, anhydrous colloidal silica, solid semi-synthetic glycerides.

Product description

Each box contains 2 blister packs of 5 ovules each.

Method of Use

Wash your hands thoroughly before opening the package.

Take one ovule from the blister pack, trying not to damage the pockets of the other remaining ovules.

Gently insert the ovule into your vagina, trying to insert it high into the vagina with the help of your fingers to avoid accidental leakage.

Frequency of Use

Unless otherwise specified by your doctor, it is recommended that you apply 1 ovule inside the vagina once a day. It is recommended that you apply the ovule in the evening before bed to ensure the best action of the device.

Do not use **Ialuril® Ovules** for prolonged periods. If symptoms of irritation persist, consult your doctor.

Warnings and precautions for use

Ialuril® Ovules are for vaginal use only.

Before using **Ialuril® Ovules**, ask your doctor for advice if you suffer from atopic allergy.

Do not expose to temperatures exceeding 25 °C.

Do not use **Ialuril® Ovules** after the expiry date, which is stated on the label.

Do not use **ialuril® Ovules** if the packaging is open or damaged.

Once used, dispose of the product in accordance with normal separate waste disposal requirements.

KEEP OUT OF THE REACH OF CHILDREN

Pregnancy

Not for use during pregnancy, unless otherwise directed by a doctor.

Contraindications

At present, there are no known contraindications for the use of **Ialuril® Ovules**.

Do not use in the case of known hypersensitivity to one of the ingredients.

Do not use in patients allergic to Compositae.

Undesirable Effects

There are no undesirable effects resulting from the use of **Ialuril® Ovules**.
Please tell your doctor or pharmacist if you experience any side effects while using the device.

Expiry date: 36 months.

The expiration date indicates the shelf life of the medical device

DATE OF LAST REVISION OF PACKAGE LEAFLET: June 2020

Year of CE certification: 2010

Manufacturer

IBSA FARMACEUTICI ITALIA srl
Via Martiri di Cefalonia, 2
26900 Lodi (LO), Italy
info@ibsa.it

Distributor


(Name and address of Distributor)



Disposable



Caution: read the warnings carefully

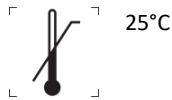

Upper limit of temperature



Lot



Use by



Do not use if the packaging is damaged



Consult the instructions for use